

2627 Redwing Rd
STE 230
Ft. Collins, CO 80526

Shipping Kit for BMA/BMC and WB/PRP Specimens



Contents:


1. Sample Vials
2. Biohazard Bag
3. Foam Shipper Box w/Rubber Band
4. FedEx Clinical Pak
5. FedEx Shipping Label Pouch
6. Requisition Form
7. Overnight Shipping Label (request via email as needed)

To have kits set to your clinic, contact Ryan Dregalla (rdregalla@DreMedTech.com; 720.201.2666) or Jessica McCluskey (jmcccluskey@DreMedTech.com; 661.714.4947)




Load sample in vial

A minimum of 0.25cc of BMA/BMC is required, $\geq 0.5\text{cc}$ preferred.

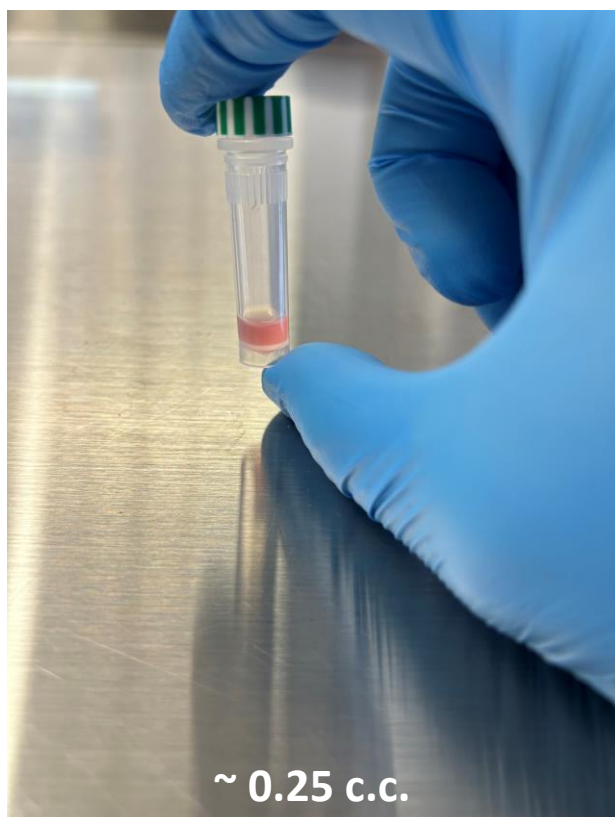


~ 0.25 c.c.


2. Ensure a minimum of 0.25cc of BMA/BMC is loaded, $\geq 0.5\text{cc}$ preferred.

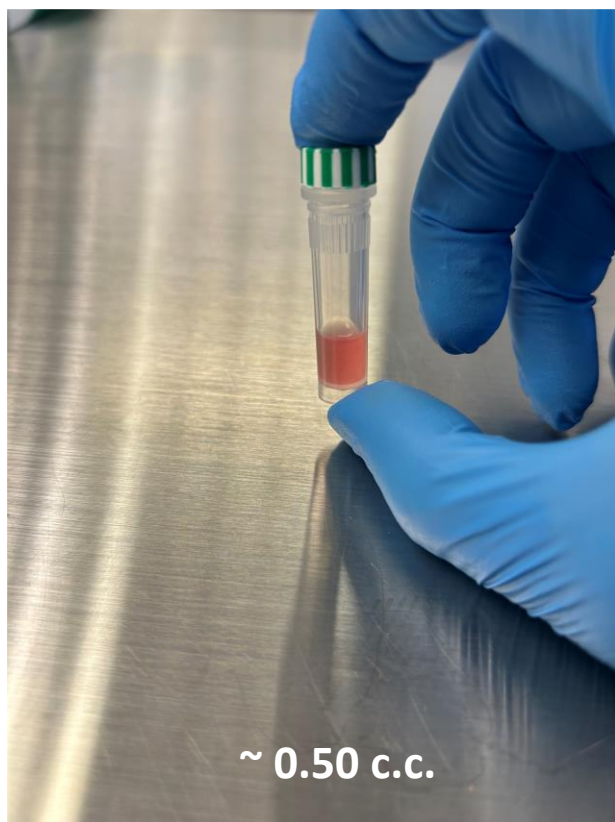

~ 0.50 c.c.

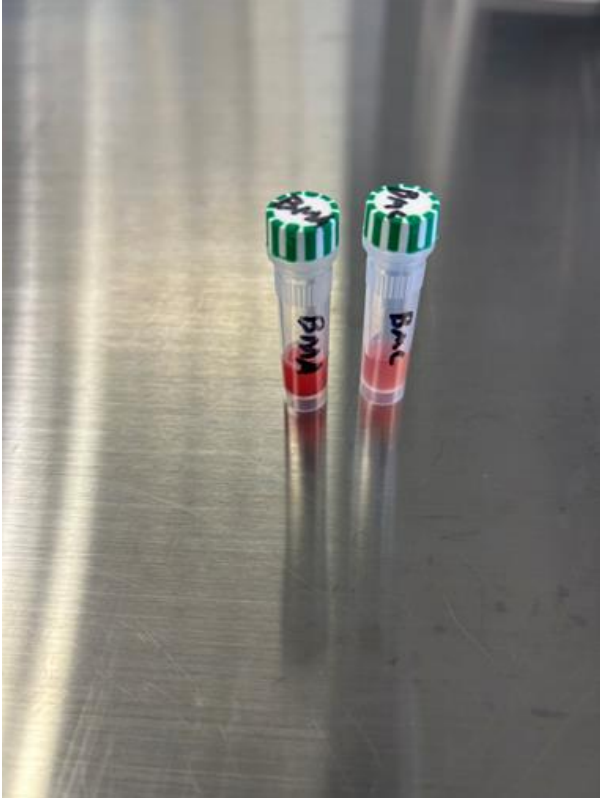

4. Label vials accordingly using a Sharpie®.

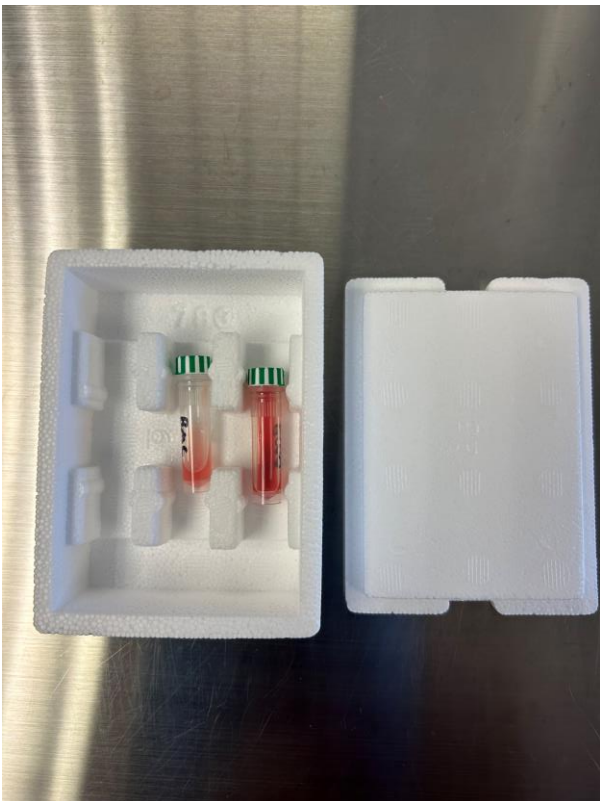

5. Place sample in foam shipper. *No cooling packs are required.* All samples are shipped ambient.



6. Secure foam shipper with rubber band.



7. Place foam shipper in Biohazard bag with a paper towel in the bag.



8. Place biohazard bag into FedEx Clinical Pak and seal with the adhesive strip integrated on Pak.

INCLUDE REQUISITION FORM

Fasten shipping label to package via adhesive pouch and drop at a FedEx Location before 4:20 PM your local time!

*FedEx overnight labels will be emailed to you directly to be printed. Overnight shipping typically costs between \$35-\$45 depending on your location in the US. We are happy to give you a quote ahead of your shipment.

rdregalla@DreMedTech.com; 720.201.2666 (Ryan)
jmcccluskey@DreMedTech.com; 661.714.4947 (Jessica)