

2627 Redwing Rd
STE 230
Ft. Collins, CO 80526

Shipping Kit for LA/MFAT Specimens



Contents:

1. Biohazard Bag
2. Insulated Foil Pouch
3. FedEx Clinical Pak
4. FedEx Shipping Label Pouch
5. Requisition Form
6. Overnight Shipping Label (request via email as needed)

*You will use a $\leq 5\text{c.c.}$ syringe and a sterile cap *from your clinic* as the vessel for containment of your MFAT specimen.

To have kits set to your clinic, contact Ryan Dregalla (rdregalla@DreMedTech.com; 720.201.2666) or Jessica McCluskey (jmcccluskey@DreMedTech.com; 661.714.4947)



1. Load sample syringe and connect with a luer-luer connector to a new receiving syringe.



2. Hold syringe vertical with the receiving syringe on top. This ensures the transfer of adipose tissue and not just saline and tumescent fluid.



3. ≥ 0.5 c.c. minimum, >0.75 c.c. MFAT preferred.

LABEL YOUR SYRINGE TO IDENTIFY CONTENTS.



4. Place syringe in biohazard bag. Please include a paper towel to absorb any leaks/spills.



5. Place sample in foil insulated pouch. *No cooling packs are required.* All samples are shipped ambient.



6. Please fold and tuck the flap into the pouch. It is NOT necessary to peel the adhesive on the pouch, as this is sealed in a secondary pouch (FedEx Clinic Pak).



7. Place foil insulating pouch in the FedEx Clinical Pak and seal with the adhesive strip integrated on Clinic Pak.

INCLUDE REQUISITION FORM



8. Fasten shipping label to package via adhesive pouch and drop at a FedEx Location before 4:20 PM your local time!

*FedEx overnight labels will be emailed to you directly to be printed. Overnight shipping typically costs between \$35-\$45 depending on your location in the US. We are happy to give you a quote ahead of your shipment.

rdregalla@DreMedTech.com; 720.201.2666 (Ryan)
jmcccluskey@DreMedTech.com; 661.714.4947 (Jessica)